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[EN] Consumer Information

A flexible wound dressing for the prevention and treatment of Radiation Dermatitis.

When used as directed StrataXRT dries to form a protective layer that is gas permeable and waterproof which hydrates and protects compromised skin areas and superficial wounds. StrataXRT may be applied to radiation dermatitis, compromised skin surfaces, superficial wounds, first and second degree burns and superficial radiation therapy burns.

How StrataXRT works:

StrataXRT dries to form a thin, flexible wound dressing which hydrates and protects affected areas. StrataXRT is inert and has no measurable pH value so does not affect the fragile acid mantle of the skin. StrataXRT helps to promote a moist wound healing environment for widespread superficial wounds and burns. StrataXRT is a self-drying, non-sticky, transparent, gel formulation that lightly bonds to the most superficial damaged skin layer.

This moist wound healing environment promotes faster reepithelialisation and significantly reduces the skin's acute inflammatory response seen with both dry and wet desquamation from radiation dosing and first and second degree burns.

In dry desquamation, StrataXRT protects the fragile epidermal covering whilst optimizing the environment for tissue healing and cellular repopulation during repetitive insult.

In wet desquamation, StrataXRT protects the dermal stroma from long-term deterioration whilst optimizing the environment for the reparative process and reducing the risk of infection.

StrataXRT is ideal for large surface areas and contoured skin like the breasts, face, back and pelvic area, as well as joints and hairy areas without the need for shaving.

Intended use:

StrataXRT is used to relieve low grade inflammatory changes such as dry, itching, flaking, peeling and irritated skin. For more severe inflammatory changes, StrataXRT reduces pain, redness and heat and helps soothe the exposed skin areas.

StrataXRT was developed for use on all types of radiation dermatitis, toxic and compromised skin, superficial wounds and burns, including those resulting from:

- Radiation therapy
- First and second degree burns

StrataXRT may be applied to superficial wounds, burns and areas where skin integrity has been compromised. StrataXRT may also be used in conjunction with other adjunctive treatments to improve overall results. StrataXRT contains no alcohols, parabens or fragrances and is transparent and odorless. StrataXRT can be used with or without a secondary protective dressing. StrataXRT is suitable for children and people with sensitive skin.

Directions for use

- Ensure that the affected skin or superficial wound is clean and dry.
- Gently pat dry as much excess exudate or wound fluid from the area as possible prior to gel application.
- Apply a very thin layer of StrataXRT directly to the affected area and allow the gel to dry.
- StrataXRT should be applied once or twice daily to affected areas or as advised by your physician.
- Once dry, StrataXRT may be covered by sun screen, cosmetics and clothing.
- For best results StrataXRT should be maintained in continuous contact with the skin (24hrs/day).

When applied correctly to the affected areas, StrataXRT should be dry in 5–6 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

Recommended duration of treatment

StrataXRT is recommended as prophylaxis following the initial radiation dose and should continue to be applied for a minimum of 60–90 days (24/7) post radiation therapy, or until no further improvement is seen. For chronic radiation dermatitis, continued use is recommended until no further improvement is seen.

How much StrataXRT do I need?

StrataXRT gel is a unique formulation that requires substantially less product per application than typical moisturizing creams or barrier ointments.

Size 50 g (1.75oz) is enough to treat an area of 12 cm × 15 cm, (5 inch × 6 inch) twice per day for over 40 days.

Caution

- StrataXRT should not be placed in contact with the eyes.
- StrataXRT should not be applied over other skin treatments without the advice of your physician.
- StrataXRT may stain clothing if not completely dry.
- StrataXRT does not contain anti-infective ingredients.
- Should your radiation dermatitis show signs of infection or failure to heal, consult your physician.
- For external use only.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for highly exudative, tunnelling wounds or 3rd degree burns.
- Keep out of the reach of children.
- Store below 30 °C (86 °F).
- Do not use after the expiration (EXP) date printed on the tube.

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Stratpharma AG, CH-4051 Basel